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INSTRUCTION MANUAL ELECTRIC SCOOTER LIBRA

NOTICE

Notice to the user and/or patient: Any serious incident that has occurred in connection to the product must be communicated to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

INDICATIONS

- Only one user can ride the scooter.
- The lowest speed must be used when carrying out the following actions: Going up, going down or crossing a slope or sloping terrain and also on uneven ground, ramps and soft surfaces.
- Keep away from flammable objects while charging the device.

CONTRAINDICATIONS

- Never try to go up or down a slope when the surface is slippery or there is a risk of skidding.
- Do not ride on a slope over 6 degrees.
- Avoid shifting the center of gravity when turning left or right.
- Direct exposure to rain or humidity will stop the scooter from functioning properly, it may cause the scooter to oxidise.
- Never put the scooter in neutral when you are on a slope.
- Do not try to modify or manipulate the scooter in regards to its mechanism or structure.
- Do not ride the scooter on stairs or escalators.
- Do not use the device while it is charging.

The Mobiclinic Libra scooter helps to solve, in a comfortable and autonomous way, mobility problems for people who have difficulty walking. The Libra scooter is designed for indoor and outdoor use.

OPERATING YOUR SCOOTER**You can make the following adjustments to increase your comfort when driving:**

- Remove rear shroud and connect the battery wire.
- Adjust seat height and location
- Adjust steering column to comfortable position.
- Adjust legroom

1. Before operating your scooter, check the following:
 - free-wheeling device on D -speed dial is at the turtle picture.
2. Sit on scooter and turn on key. The Battery Gauge light should be lit in the green, yellow, and red color zones. The Self-Diagnostic Warning Light should not be blinking.
3. When your hands rest comfortably on the handlebars, the thumb control levers should be within easy reach. The right lever moves the scooter forward, the left one moves it backward. When you release both levers the scooter will stop.
4. Steer the scooter by turning the whole steering column the way you want to go.
5. Practice driving where there are no obstacles. Start at the slowest speed and drive forward and backward; make some turns. As you get more comfortable you can increase the speed by turning the speed dial toward the picture of the rabbit.
6. If two Battery Gauge LED Lights be lit, you should plan to recharge the batteries very soon.
7. If the scooter stops, locate the circuit breaker on the top of the battery pack. Push it and try the scooter again.
8. When you are finished riding, turn off the key before getting off.
9. If you are finished riding for the day, immediately recharge the batteries.

OTHER OPERATING INFORMATION


Hill climbing: You may need to use a higher speed. Turn to lower speed before going downhill.

Down slopes: To proceed down steep slopes slowly, set speed control in proximity of the turtle. This enables driver control, as the closer the speed control is set toward the turtle, the slower it will become. However, this scooter will not self-accelerate down hills due to the automatic braking taking effect should you attempt to drive too fast.

If the scooter breaks down and must be moved, get off the scooter, push the Free-Wheeling Lever to N, move the scooter slowly to a safe location, and move the lever back to D.

IDENTIFYING THE PARTS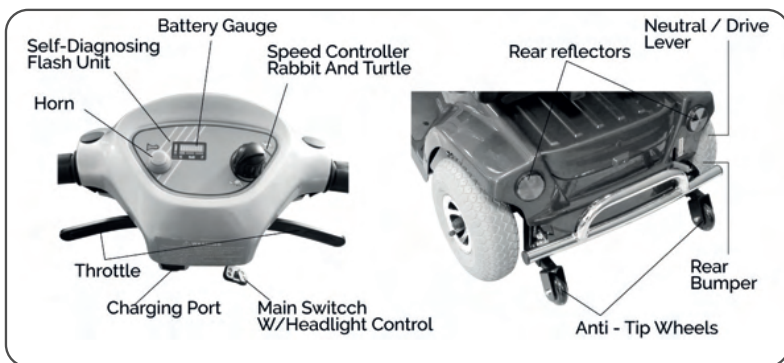

FUNCTION OF PARTS

Top control panel

1. Speed Control Knob: The Rabbit means fast and Turtle is slow. By turning this, you control the total speed transferred to the thumb controls.
2. Self Diagnostic Warning Light: Flashing indicates problems with the scooter.
3. Battery Gauge: This light is yellow. Below you will find three color ranges: red, yellow and green. When the light is on in all three color ranges, the batteries are fully charged. When the light is on only in the red, or red and yellow areas, the batteries need to be recharged.

Beneath top control panel

1. Throttle: Pushing the right Throttle moves the scooter forward. Pushing the left Throttle moves the scooter backward. (This can be reversed if required by local dealer.) Releasing both engages the automatic brake. This is also your accelerator. The further you depress them, the faster you go. (Subject to the position of the Rabbit/Turtle control).

At base of steering tiller

1. Adjust the steering handlebars: Push down on the handlebars to adjust them. Pull up to lock the turn of the steering column at an angle that is comfortable for you.

Under the seat

1. Seat adjustment lever:

- Handlebar clearance adjustment: pull up on the lever on the front of the seat to adjust the handlebar clearance.
- Rotation adjustment: press the lever backwards to rotate the seat; press it forwards to lock it in the desired position.
- Armrest adjustment: Loosen the two thumbscrews to adjust the width between the armrests. Tighten them again to secure them in the desired position.

Rear body

1. Rear Cover: Protects batteries from damage.

2. Anti-tip wheels: Helps keep the scooter from tipping over.

3. The Free-Wheeling Lever: It will let you change the Mode of your scooter (Neutral Mode or Drive Mode). The Neutral Mode (N) will let you move the scooter without energy, you will be able to push it, whereas the Drive Mode (D) will let you drive the scooter and immobilize it when you want to park it.

You will find some stickers next to the lever, they will help you to know the Mode of your scooter.

If you would like to change from Neutral Mode to Drive Mode, you must turn off the scooter. After changing the Mode, you will be able to turn on again your scooter.

ASSEMBLING YOUR SCOOTER

To assemble the scooter, you can repeat the disassembly directions in reverse. Abbreviated directions are given below:

1. Use the Steering Column Lever to move the Steering Column up out of the way.

2. Tilting the rear end to a horizontal position, insert the scooter frame tube of the front end into the rear section first, making sure that the locking pins are removed.

3. Slide the two sections of the unit together until the front half reaches its limit.

4. Insert the two locking pins into their corresponding holes. Both locking pins must be fully inserted whenever the scooter is assembled.

5. Place batteries, use Velcro to fix the position and attach the battery power plugs.

6. Place rear shroud.

7. Screw together controller wiring harness.

8. Place rear storage cover.

9. Loosen Seat Rotation Lever counterclockwise. Place seat on seat post. Tighten Lever again by turning clockwise. Lock Seat Control Lever on bottom of seat before sitting down.

ELECTROMAGNETIC INTERFERENCE AND WARNINGS

Powered wheelchairs and motorized scooters may be susceptible to electro- magnetic interference (EMI), which is interfering electromagnetic energy (EM) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones. The interference (from radio wave sources) can cause the motorized scooter to release its brakes, move by itself, or move in unintended directions. It can also permanently damage the motorized scooter control system.

1. Do not operate hand-held transceivers (transmitters-receivers), such as citizens band (CB) radios, or turn ON personal communication devices, such as cellular phones, while the motorized scooter is turned ON.

2. Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them.
3. If unintended movement or brake release occurs, turn the motorized scooter OFF as soon as it is safe.
4. Be aware that adding accessories or components, or modifying the motorized scooter, may make it more susceptible to EMI.

20 volts per meter (V/m) is a generally achievable and useful immunity level against EMI (as of May 1994). The higher the level, the greater the protection. This product has an immunity level of at least 10 V/m.

CHARGING THE BATTERIES

Batteries must be charged before using the scooter for the first time and should be charged after each day use. You will need scooter and the battery charger.

1. Insert battery charger cord into the charger connector on the charger output. Refer to above figure for correct position
2. Plug the other end of the battery charger cable into a standard electrical outlet.
3. Turn the power on. Normally, The LED (Power) Light will turn on when electric current passes.
4. Charging starts. During charging, LED (Charge) will indicate orange light, when it turns to green light, that means well-charged.

5. LED indication

- LED(POWER) - GREEN LIGHT ON: Power On.
- LED(CHARGE) - ORANGE LIGHT ON: Charging / GREEN LIGHT ON: Fully Charged

6. Charger Trouble Shooting

(A) If LED (POWER) light is off.

- Check the input voltage (115V/230V) is the same as you adjust.
- If light is still off, please check and repair the battery charger.

(B) If LED (CHARGE) light is off.

- Check to see clips connection is correct.
- If the battery is fully charged, the LED (CHARGE) light will be off.
- If light is still off, the battery may be defective.

(C) If ORANGE light can turn to GREEN.

- The battery can not be charged. Please check and recover it.

(D) If ORANGE light turns to GREEN immediately


- Check to see the battery is fully charged, if not, The battery may be defective Check and recover it.

Batteries must be fully charged in order to not hurt their lifetime. If not, faults may result


DISASSEMBLING YOUR SCOOTER

NO TOOLS ARE NECESSARY. YOU CAN DISASSEMBLE SCOOTER FOR TRANSPORT


1. Remove seat by releasing Seat Rotation Lever (A) and then lift off. Gently lift rear storage cover (B).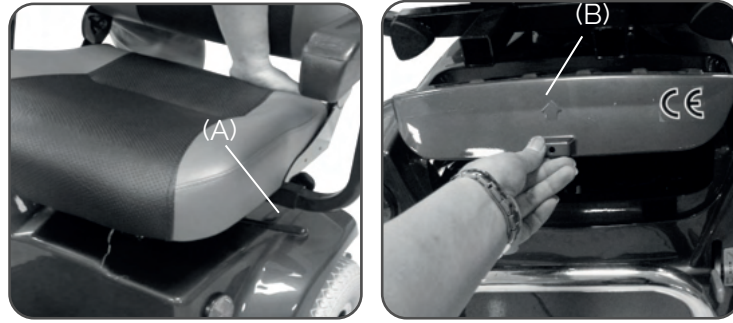


Unscrew controller wiring harnesses (C). Remove rear shroud off of the scooter (D).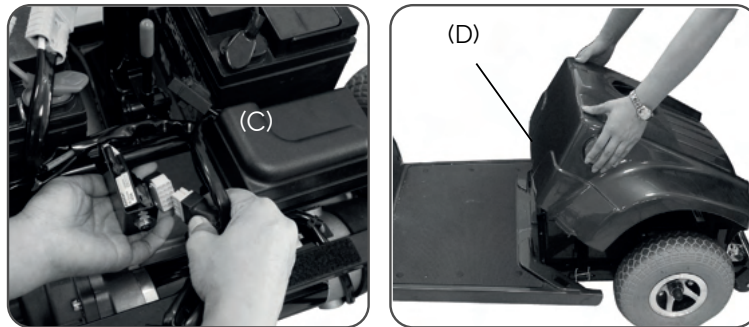

Remove battery fixing Velcro (E). Unplug both battery harnesses (F) and remove batteries (G).

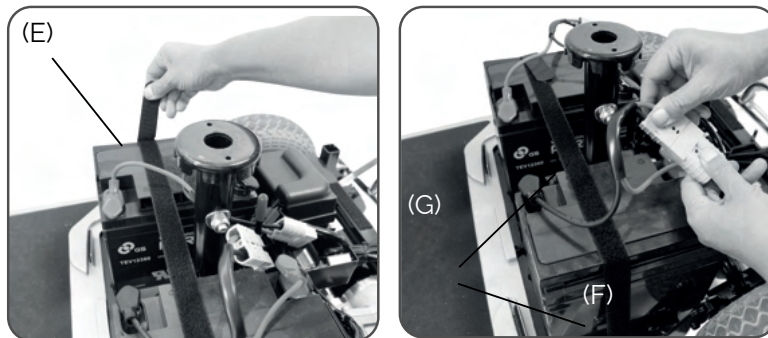
Detach the front and rear sections by holding the Seat Post (J) with one hand and the other hand pushing the release lever to the rear of the scooter. Lift the seat post to separate the Rear Frame Assembly (H) from the Front Frame Assembly.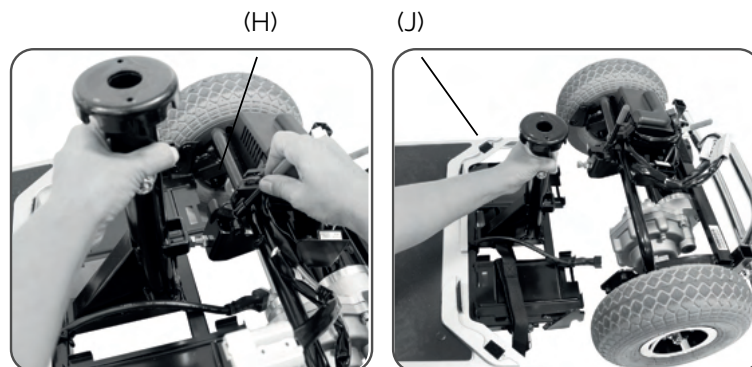

Fuse Replacement

1. Remove the front cover screws (R), the arrow indicates the direction. (Caution, the screws have different sizes).
2. Then open the box where the fuses are located (R1) and replace it with a backup fuse (R2), the fuse must be of the same rating.

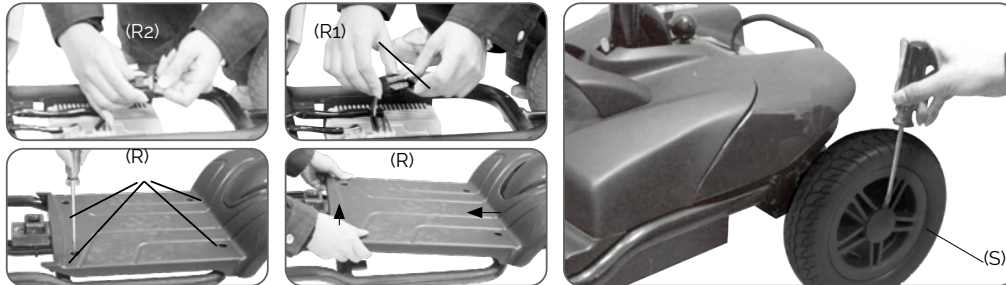CLEANING AND MAINTAINING YOUR SCOOTER

If your scooter is dirty, use a damp or lightly soapy cloth to wipe it down. Do not use running water to wash or rinse the scooter in order to protect the electrical parts. Polish with an automotive liquid polish.

All maintenance and repair of your scooter should be done by an authorized dealer.

The following areas required periodic inspection:

- Tire pressure between 30-50 psi.
- All electrical connections are firmly attached.

Specifications	
Overall Length	128 cm / 50.4"
Overall Width	61cm / 24"
Overall Height	111 cm / 43.7"
Front Wheels	26 cm / 10"
Rear Wheels	26 cm / 10"
Weight W/ Batteries	87.9 kg / 194 lbs
Max. Speed	8 kmph / 5 mph
Weight Capacity	135 kg / 300 lbs
Ground Clearance	4 cm / 1.6"
Grade Climbable	12 degree
Curb Climbable	7 cm / 2.75"
Turning Radius	152 cm / 59.8"
Brake	Electro-Mechanical
Seat Type	Swivel Mid-Back W/ Headrest & Seat Sliding Mechanism
Seat Width	45.5 cm / 18"
Motor Size	500W, 3400 r.p.m
Battery Size	(2) 12V. 36Ah
Weight of Battery Pack	22.5 kg / 49.6 lbs
Travel Range	30 km / 18.6 Miles
Battery Charger	5A Off Board

TROUBLESHOOTING

To check the Self-Diagnostic Warning Light, turn on the key and count the number of flashes

Flash	Description	Meaning
1	Low Battery	The batteries are running low. •Recharge the batteries
2	Low Battery Fault	Battery charge is empty. • Recharge the batteries.
3	Battery voltage too high	Battery voltage is too high. This may occur if overcharged &/or traveling down a long slope. • Turn the scooter power off then on again • If traveling down a slope, reduce your speed.
4	Current limit time-out	The motor has been exceeding its maximum current rating for too long. This may be due to a faulty motor • Check the motor and associated connections and wiring. • Turn the controller off, leave for a few minutes and turn back on again.
5	Brake fault	Either a park brake release switch is active or the park brake is faulty. • Check if the lever is in Automatic Mode (D). • If this flash code does not appear until the throttle has been moved out of neutral, check for a short in the park-brake circuit. • If this flash code appears at power-up, check for an open circuit in the park-brake circuit. • Check the park brake and associated connections and wiring
6	Out of neutral at power-up	The Throttle is out of neutral when turning the controller on. • Ensure the throttle is in neutral when turning the scooter on. • The Throttle may require re-calibration. • Check the its associated connections and wiring.
7	Speed Pot Fault	The throttle, speed limit pot or their associated wiring may be faulty. • Check the throttle and speed pot and associated connections and wiring.
8	Motor Voltage Fault	The motor or its associated wiring is faulty. • Check the motor and associated connections and wiring.
9	Other error	The controller may have an internal fault. • Check all connections & wiring.

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CHIEN TI ENTERPRISE CO., LTD.
GRUPO R. QUERALTÓ S.A.