MANUAL DE INSTRUCCIONES

Colchón antiescaras MOBI 2

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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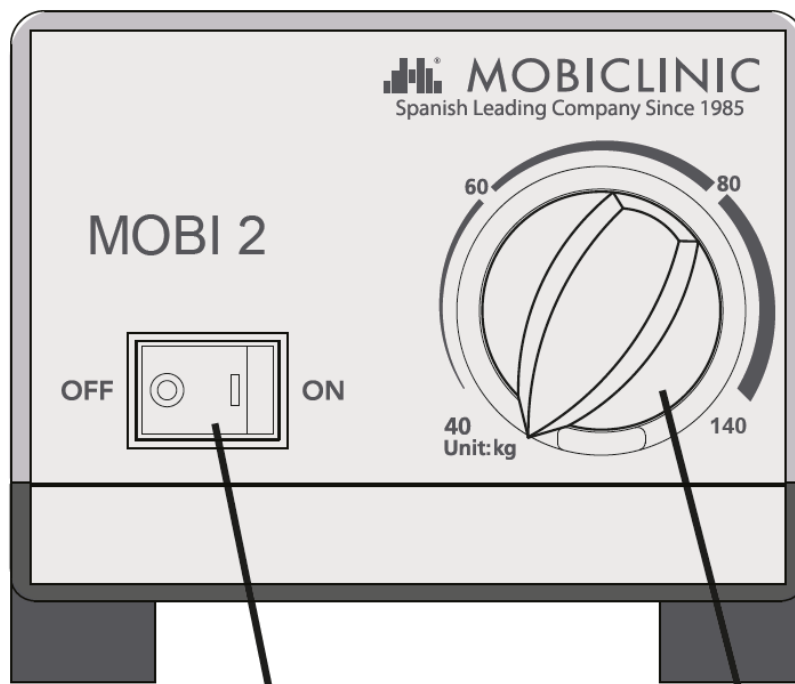
Apagar / Encender

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

This device consists of two parts: the mattress and the motor. Thanks to its small compressor, the motor generates little noise and consumes little energy. The control panel is simple and very easy to use. The mattress relieves pressure through its alternating air cells that inflate and deflate approximately every 12 minutes. Constant pressure on a bony prominence is known to be the main cause of bedsores. The continuous movement of the mattress decreases the constant pressure and improves circulation.

1. USE

1. Place the mattress on the bed with the air sleeve towards the feet.
2. Use the integrated hooks to hang the compressor securely on the footboard of the bed. You can also place it on a flat surface.
3. Connect the air hoses of the mattress to the compressor.
4. Connect the compressor to an electrical outlet and ensure that the cable is not in danger of damage.
5. Turn on the switch on the compressor control panel to start inflating the mattress.
6. Once inflated, adjust the mattress using the compressor knob.

2. INDICATIONS

This product is intended for:

1. To assist and reduce the incidence of pressure ulcers by optimising patient comfort.
2. For long-term home care of patients suffering from pressure ulcers.
3. For the treatment of pain according to medical prescription.
4. If necessary, replace the fuse with the one indicated: T1A 250V. Input current: ~220V 50Hz, 0.1A.
5. Supported weight: 140 kg.

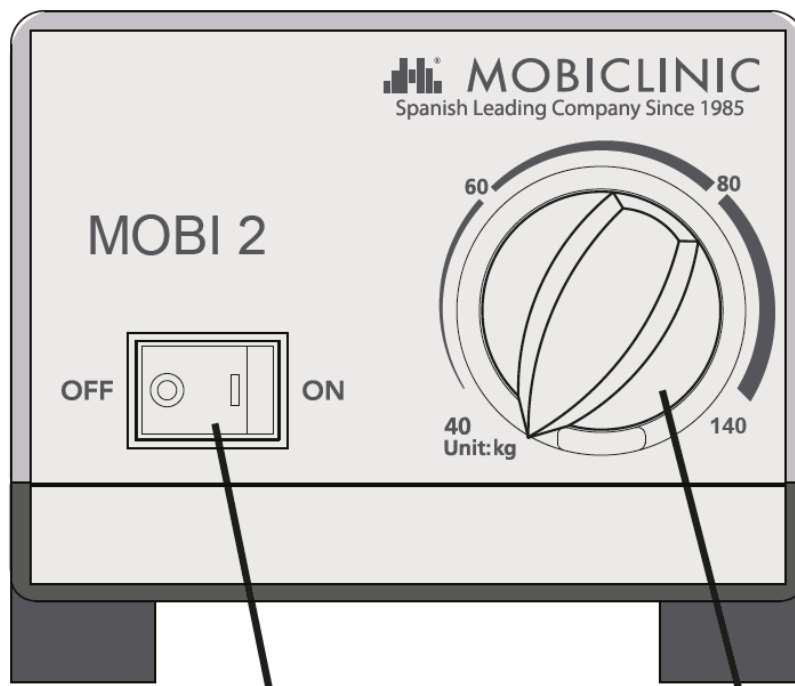
3. CONTRAINDICATIONS

1. This equipment is not suitable for use in the presence of a flammable anaesthetic mixture with air or with pure oxygen or nitrous oxide.
2. Do not turn or roll patients with a bone fracture or spinal injury.
3. Do not smoke near the compressor.
4. Do not bring the compressor close to hot surfaces.
5. Risk of explosion if used near flammable anaesthetics.

4. CLEANING AND MAINTENANCE

1. Make sure that the system is disconnected from the mains before cleaning.
2. Do not immerse in water.
3. Do not dry clean.
4. Do not use bleach or products containing bleach.
5. Do not iron.
6. Do not autoclave or use phenol for the cleaning procedure.

7. Do not wash air cells, clean with mild soapy water or disinfectant.
8. Environmental conditions:
 - 8.1. Operating conditions:
 - Ambient temperature: 10°C to 40°C.
 - Relative humidity: 10% to 75%.
 - 8.2. Storage and shipping conditions:
 - Ambient temperature: From -18°C to 43°C.
 - Relative humidity: 10% to 95%.

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GUANGDONG YUEHUA MEDICAL INSTRUMENT FACTORY CO., LTD
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.