MANUAL DE INSTRUCCIONES

Silla de ruedas Venecia

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

HOW TO USE

Various activities require the user to reach, lean or transfer from the chair to another seat or vice versa. These movements can cause a change in the balance, centre of gravity and weight distribution of the wheelchair. The design of the wheelchair will allow for stability in daily activities when used correctly by following the recommendations:

1. Reaching for objects / Leaning forward: Move the wheelchair as close as possible to the object you want to reach. Apply the brakes to lock the rear wheels. Only attempt to grasp objects that are within arm reach while in a normal seated position. Only your arms should extend out of the seat of the wheelchair.

2. Reaching for objects / Leaning Back: Move the chair as close as you can to the object you want to reach. Only reach as far as your arm will allow without changing the normal sitting position. Do not lean over the top of the backrest, as this may cause the chair to tip over.

3. Reaching / leaning sideways: Move the chair as close as possible to the object you want to reach. Turn the front wheels so that they point in the direction you are going to lean towards. Apply the brakes on the rear wheels. Do not shift your weight to the side you are leaning towards or you may tip your chair over.

4. Transferring from one seat to another: First, make sure that the space between the two seats is the least possible. Confirm that the brakes are applied. Turn the front wheels towards the direction you want to move to. Lift the footrests and fold the supports sideways. It is important to have professional supervision until the user is familiar with the process.

5. Overcoming obstacles

5.1. Climbing a kerb from the front: The attendant should lean the wheelchair back until the front wheels are over the kerb. Move the wheelchair forward until the rear wheels touch the kerb and carefully lower the front wheels. Lift and push until the rear wheels are over the kerb. The wheelchair should not be lifted by the handlebars alone; use your foot to press down on the rear of the frame.

5.2. Climbing a kerb backwards: the wheelchair must be positioned so that the rear wheels are close to the kerb. The attendant should lean the wheelchair back and pull the wheelchair until the rear wheels come up to the kerb. The front wheels should only be lowered when the wheelchair is at a safe distance from the edge of the kerb.

5.3. Going up and down slopes: Always go straight up and down slopes to avoid the risk of tipping over.

INDICATIONS

- The wheelchair is used as a mobility tool for the disabled, patients and elderly that have difficulties in action.
- Beginners must be accompanied by a partner. to avoid accidents due to improper use.

CONTRAINDICATIONS

- Before using the product, ensure that the parts are secure, the wheels are in working condition, the wheelchair is balanced and all wheels are in contact with the ground at all times during use.
- Be aware of the risks when reaching for objects, leaning backwards or sideways, the use of manual brakes, the wheels and the weight limit.
- Footrests are designed to support your feet while seated. Do not stand on them.
- Do not use the wheelchair in moving vehicles.

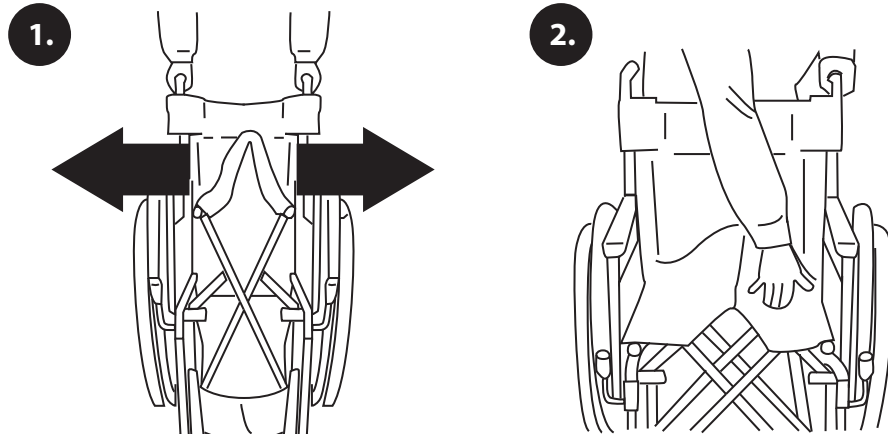
PARTS


1. Backrest: You can fold the backrest by pressing down on the two black plastic levers, then pulling back and pushing down on the top of the backrest. To return it to its original position, repeat the operation in reverse, making sure that the mechanism is securely locked.
2. Handles.
3. Armrests.
4. Seat.
5. Footrest support.
6. Footrest: you can adjust the height of the footrest by loosening the nut at the base of the adjustment tube, positioning the plates at the desired height and tightening the nut to secure them.
7. Wheels and bearings: The wheels are fitted with puncture-proof tyres. The swivel fork allows the wheelchair to be steered. An adjustment locknut is located under the fork cap. If the lock nut is too tight, the wheelchair will be difficult to steer.
8. Brakes: they must always be activated at the same time. To do this, pull the lever upwards. To disengage them, push the lever downwards.
9. Cross bar.
10. Lever for folding the backrest.

ASSEMBLY

1. To unfold the chair, press down on the seat side tubes until the chair is fully unfolded. Make sure that the seat rails are securely engaged in the frame brackets before sitting down.

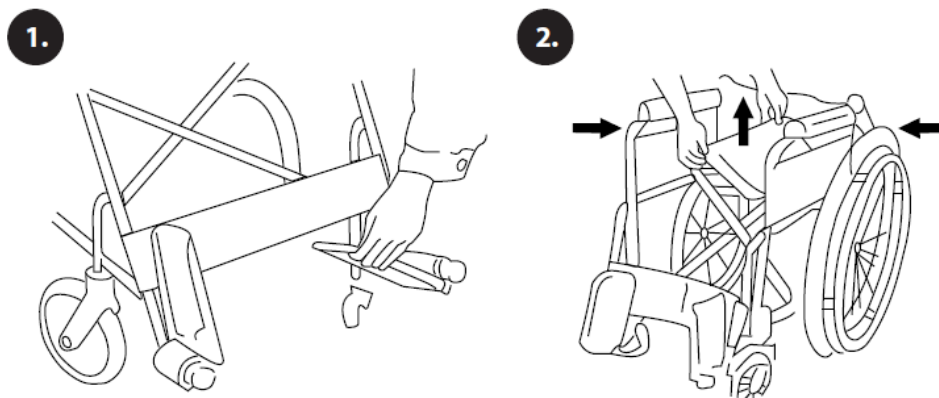2. To fold the chair, pull the front and rear side of the seat fabric upwards.

3. Footrest support.

3.1. Place it on its side so that the footrest is perpendicular to the chair.

3.2. Push the brackets towards the chair until they lock into place. The footrest will face inwards towards the chair when it is locked into place.

3.3. To remove, push the lever inwards and rotate the footrest outwards.

4. Rear wheels.

4.1. Grasp the rear wheel and press the quick release button.

4.2. While pressing the button, push the axle against the wheelchair until you hear a click.

4.3. Check that the wheel is securely fixed.

4.4. Follow the reverse process to remove the wheels.

ADJUSTING THE HEIGHT OF THE WHEELCHAIR

To adjust the height of the wheelchair, please follow the steps below:

1. Remove the rear wheel by pressing the quick release button.
2. With the help of a spanner (**not included**), remove the larger inner nut.

IMPORTANT: For the main nut a wrench according to its size must be used for correct tightening. The tool provided is not the correct one for this step.

3. Change the height by adjusting the two bolts to the desired distance with the tool provided.

MAINTENANCE AND CLEANING

1. Upholstery: check the upholstery regularly to ensure that there are no tears or looseness. It should be cleaned regularly with a mixture of neutral soap and water. Do not use solvents.
2. Metal parts: Clean with wax products, such as car wax, to preserve the finish. Do not use abrasive cleaners that may damage the varnish.
3. Cross bar: Inspect periodically and lubricate the centre bolt with light machine oil.
4. Wheels and tyres: Inspect the wheels for wear or cracks. It is recommended that both front wheels be replaced at the same time. For tyres, check that the tyre pressure is correct and both tyres are even.
5. Bearings: The front wheels are attached to the swivel fork by means of a bolt and axle nut. They should be checked and tightened periodically, adjusting them so that they are secure but free to move.
6. Brakes: check frequently that they are working properly and are not worn.
7. Check that the wheelchair folds and unfolds properly and that the wheelchair moves in a straight line.

TECHNICAL SPECIFICATIONS

Frame	Single cross brace, liquid painting
Front wheels	6"x1.25", PU tyres
Back wheels	24", spoke wheel, PU tyres
Armrests	Adjustable height, PU pad
Footrests	Foldable, detachable, plastic plates
Seat and backrest	Nylon with foam
Front fork	Aluminum, adjustable height
Backrest	Foldable

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INTCO MEDICAL (HK) CO., LTD
GRUPO R. QUERALTÓ S.A.