# Aquatec® 90 Ergo

en	Raised Toilet Seat User Manual .....	2
de	Toilettensitzerhöhung Gebrauchsanweisung .....	4
fr	Réhausse WC Manuel d'utilisation .....	6
it	Rialzo WC Manuale d'uso .....	8
es	Asiento elevador de inodoro Manual del usuario .....	11
pt	Elevador de sanita Manual de utilização .....	13
nl	Verhoogde toiletzitting Gebruiksaanwijzing .....	15

<p style="text-align: center;">1</p>	<p style="text-align: center;">2</p>	
<p style="text-align: center;">3</p>	<p style="text-align: center;">4</p>	<p style="text-align: center;">5</p>
<p style="text-align: center;">6</p>	<p style="text-align: center;">7</p>	

**AQUATEC®**

This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and saved for future reference.

**Yes, you can.®**

# 1 General

## 1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions. If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website.

Invacare reserves the right to alter product specifications without further notice.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

## 1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

## 1.3 Compliance

This CE- and UKCA-marked medical device conforms to: EU Medical Device Regulation, UK Medical Device Regulation, ISO 13485, ISO 14971, REACH, ISO 17966 and related standards.

## 1.4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## 1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up
- Unauthorised modifications and/or use of unsuitable spare parts

# 2 Safety

## 2.1 Safety Information

### **WARNING!** **Risk of Serious Injury or Damage**

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

### **WARNING!**

#### **Risk of Falling!**

- Do not use if defective.
- Do not make any unauthorised alterations or modifications to the product.
- Do not stand on this product.
- Do not use on toilet seat.
- Do not sit on the lid.
- Do not slide over from/to the side, for example from/to a wheelchair.

### **WARNING!**

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is correctly positioned.

### **CAUTION!**

#### **Risk of Injury**

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalogue or website for available options or contact your Invacare distributor. See addresses at the end of this document.

## 2.2 Labels and Symbols on the Product

### Identification Label (Fig. 6)

Ⓐ	Manufacturer	Ⓒ	Production date
Ⓑ	UK Conformity Assessed	Ⓓ	Unique Device Identifier
Ⓒ	European Conformity	Ⓔ	Medical Device
Ⓓ	Lot number	Ⓕ	Reference number
Ⓔ	Read user manual	Ⓖ	Product designation
Ⓕ	Maximum user weight		

The identification label is attached to the back of the toilet seat.

# 3 Product Overview

## 3.1 Intended Use

The Aquatec 90 Ergo is a raised toilet seat that elevates the seating height on toilets for 10 cm for a better seating position. The product is fixed by simply placing it onto the toilet bowl. The product sticks on the toilet bowl due to its adhesive material underneath.

The product is intended to be used as an entry level aid for self-hygiene care and toilet purposes. The product provides support for sitting down or standing up from a toilet.

### Intended Users

Users are any adults and adolescents with limited mobility or limited muscle strength. The user must be able to keep own balance or ensures the availability of additional support i.e. a wall mounted grab bar.

### Indications

- Limited mobility (e.g. legs and/or hips) but adequate body stability or
- having muscle strength issues to stand up/sit down due to transitional or permanent disabilities.

## Contraindications

There are no contraindications known when using the product as intended. For the maximum user weight of the product, see 7 *Technical Data*, page 3 .

## 3.2 Scope of Delivery

The scope of delivery is shown in figures 1 & 2.

## 4 Setup

1. Install the lid (if included) by clicking the lid into the holders (Fig. 3).
2. Flip up your toilet seat (Fig. 4). (Alternative: Remove toilet seat.)
3. Clean and dry the surface of the toilet bowl.
4. Place the raised toilet seat in the middle of the toilet bowl (Fig. 4 & 5).

### **WARNING!** **Risk of Falling!**

If the raised toilet seat is not placed in the middle of the toilet bowl, tipping over can cause injury or damage.

- Make sure that the raised toilet seat is placed in the middle of the toilet bowl, refer to Fig. 5 (label on front of the product).
- Ensure that both the raised toilet seat and toilet bowl surface are dry. Do not place the raised toilet seat on the toilet bowl if either surface is wet (Fig. 4).

## 5 Maintenance

### 5.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

### 5.2 Cleaning and Disinfection

#### General Safety Information

### **CAUTION!** **Risk of Contamination**

– Take precautions for yourself and use appropriate protective equipment.

### **NOTICE!**

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

## Cleaning Intervals

### **NOTICE!**

Regular cleaning and disinfection enhance smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- regularly while in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

## Cleaning Instructions

### **NOTICE!**

Maximum cleaning temperature:

- 60 °C by hand
- 85 °C for 3 minutes by automatic washing systems

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

## Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

## 6 After Use

### 6.1 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see 5 *Maintenance*, page 3 .

Make sure that the user manual is handed over with the product.

### 6.2 Disposal

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

## 7 Technical Data

### 7.1 Weights and Materials

Weights	
Weight model without lid	1.2 kg
Weight model with lid	1.6 kg
Maximum user weight	225 kg

Materials	
Corpus / Lid	PP

All product components are corrosion resistant and do not contain natural rubber latex.

## 7.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Austria GmbH
Invacare Poirier SAS
Invacare GmbH
Invacare UK Operations Limited