MANUAL DE INSTRUCCIONES

Bastón trípode GLORIA

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

Practical and easy to use folding seat cane. Features a curved, foam covered support handle, round PP padded seat and tripod design for extra stability. Supports a weight of up to 90kgs.

- Seat diameter: 21cm / Total height of folded product: 84cm / Height of open seat (distance from floor to seat: 50cm).

1. WARNINGS AND PRECAUTIONS

1. Do not exceed the maximum load capacity of the product.
2. Use with caution. To avoid tipping hazard, do not grasp handles with wet hands, as they may slip and lose their grip.
3. Exercise extreme caution when using on wet and slippery surfaces.
4. Walking sticks are intended only as walking aids and are not designed to support full body weight.

2. INSTRUCTIONS FOR USE

The pole is already assembled and ready for use. However, please check the condition of the components before use.

1. Never deploy and use your seat cane on a slippery, soft or uneven surface, including in bathrooms or in moving vehicles.
2. Before using the seat cane, check that all parts are properly connected and secured. If there are any signs of cracking or breakage of the cane, it should not be used.
3. Always ensure that all screws and bolts are securely fastened before use and check the stability of the pole.
4. Make sure that the folding legs are fully extended and on a secure surface. All three legs must touch the ground simultaneously.
5. Straddle the handle with your hands on the handle to maintain balance while the product is unfolded. Sit in the centre of the seat in a stable position. Do not rock or drag the chair. Do not use only two legs or a single chair leg for support.
6. Do not use the handle as a support when standing or sitting.
7. Do not hang heavy objects on the handle.
8. Do not sit sideways, recline with your back to the handle, or stand up in the chair.

3. INDICATIONS

Walking sticks are assistive devices for: patients with musculoskeletal problems or elderly people; facilitating the movement of people with disabilities, sick patients or patients with injuries and diseases of the lower limbs using orthoses or prostheses; elderly people to relieve joints; patients during a rehabilitation period after surgery; rehabilitation after fractured limbs.

4. CONTRAINDICATIONS

Very old and very young people may not have the necessary torso strength and coordination to use crutches successfully. Any musculoskeletal or neurological condition that decreases mobility, strength or sensation may be a potential contraindication to the use of crutches in the short or long term. Inability to coordinate body movements, dizziness or drowsiness.

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JIANLIAN HOMECARE PRODUCTS CO., LTD
LOTUS NL B.V.
GRUPO R. QUERALTÓ S.A.