MANUAL DE INSTRUCCIONES

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Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. INSTRUCTIONS FOR USE

To adjust the height of the chair, press the adjustment plunger on the leg and slide it to the desired height. Also ensure that the adjustment plunger is positioned securely into one of the holes. This process should be repeated for all four legs and they should all be at the same height for your safety.

2. INDICATIONS

- It is designed to assist patients, the elderly and/or the disabled.
- To be used indoors on a flat and ready surface.
- Check regularly that all nuts and bolts are tight. And make sure that all four legs are touching the ground at the same level.
- Maximum weight: 100 kg.

3. CONTRAINDICATIONS

- Only adjust, mount or dismount the wheelchair when it is not occupied.
- Do not lean your body out of the chair to prevent it from tipping over.
- Push the seat from both sides to lift it up. Do not push on one side only.
- Do not stand on the chair.

4. CLEANING AND MAINTENANCE

- Remove dirt with a damp cloth and neutral soap and then wipe dry. Do not use harsh chemicals or abrasive cleaners to avoid damaging the product.
- After use, wipe the shower stool with a dry cloth.
- Avoid direct contact with corrosive chemicals.

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GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.