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RIGID DIGITAL THERMOMETER TH-02

WARNING

Choking hazard do not swallow. Do not allow unattended children to use this device without the supervision of an adult. Do not use the thermometer in the ear. Do not place the battery close to heat sources. Remove the battery from the device if it's not going to be used for a long time. The use of temperature readings for self-diagnosis is dangerous. Consult your doctor for the interpretation of results. Do not attempt measurements when the thermometer is wet. Do not attempt to disassemble or repair the thermometer. After each use, disinfect the thermometer. Do not force the thermometer into the rectum. Stop insertion and abort the measurement if patient feels pain. If the unit has been stored at temperatures under 4° C or over 50° C, leave it in ambient temperature for about 15 minutes before using it. The device offers quick and highly accurate results. It is designed to be used orally, rectally or under the arm. It is reusable and intended for clinical or home use, on patients of all ages.

CALIBRATION

The thermometer is initially calibrated at the time of manufacture. If the thermometer is used according to use instructions, periodic readjustment is not required. However, we recommend checking calibration every two years or whenever clinical accuracy of the thermometer is in question. Turn on the thermometer and insert into the water bath and then check the laboratory accuracy of thermometer. The above recommendations do not supersede the legal requirements. The user must always comply with legal requirements for the control of the measurement, functionality, and accuracy of the device which are required by the scope of relevant laws, directives or ordinances where the device is used. The disposal of this product and the used batteries must be carried out in accordance with the national regulations corresponding to the elimination of electronic products.

DIRECTIONS

- 1. Press the ON/OFF button.
*Note: If the measured temperature is lower than 32° C (90° F), the LCD display will show "Lo", if it is higher than 42.9° C (109.9° F), it will display "Hi".
2. Place the thermometer in the desired location (mouth, rectum, or armpit).
Oral use: Place the thermometer under the tongue. Normal temperature: between 35.7° C and 37.3° C (96.3° F and 99.1° F).
Rectal use: Lubricate probe tip. Insert it approximately 1cm (less than 0.5") into rectum. Normal temperature between 36.2° C and 37.7° C (97.2° F and 99.9° F).
Armpit use: Clean and dry the armpit. Place probe in armpit and keep arm pressed firmly. This method should not be used if precise measurements are required. Normal temperature between 35.2° and 36.7° C (95.4° F and 98.1° F).
3. Wait until you hear the alarm beep. The measurement will continue, even after the beep.
4. Press ON/OFF to turn unit off after testing is complete, or the device will automatically turn off.

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WENHOU YOSUN MEDICAL TECHNOLOGY CO., LTD.
LOTUS.NL B.V.
Imported by/Importado por: Grupo B. Querol S.A.