

Última revisión: 4/11/2020

Last revision date: November 4<sup>th</sup> 2020



## User Instruction for HA-4506/ HA-4507 (Bed & Chair Risers)

### SPECIFICATION

ITEM	Overall Size (1pc)	Weight	Material	Max. Load Weight
HA-4507	140 x 140x 150 mm	677 g	PP	200 kg
HA-4506	145 x 145 x 100 mm	393 g	PP / PU Foam	200 kg

### INSTRUCTIONS FOR USE

1. Place one pair of risers next to a side of chair or bed legs.
2. Lift the chair or bed and lower the legs into the riser sockets, repeat for the other side.
3. Ensure that all the risers are level and the legs are properly located.

### WARNING

- Four units together support up to 200kgs.
- Always use the risers on a flat surface.
- The risers are not suited for recliners.
- Please make sure to distribute weight symmetrically.
- Please make sure that all the risers are level.
- Periodically inspect for wear and damage.

### MAINTENANCE

The Bed & Chair Riser are designed for minimum maintenance.

They can easily be clean with a damp cloth





## Instrucciones de uso HA-4506 / HA-4507 (Conos de elevación)

### ESPECIFICACIONES

Modelo	Tamaño total (1pc)	Peso	Material	Peso máximo soportado
HA-4507	140 x 140x 150 mm	677 g	PP	200 kg
HA-4506	145 x 145 x 100 mm	393 g	Espuma de PP / PU	200 kg

### INSTRUCCIONES DE USO

1. Coloque un par de conos junto a un lado de las patas de una silla o cama.
2. Levante la silla o cama y coloque los elevadores.
3. Asegúrese de que todos los elevadores estén nivelados.

### ADVERTENCIAS

- 4 unidades de conos de elevación soportan hasta 200kg.
- Utilice siempre los conos de elevación en superficies planas.
- Estos conos no son recomendables para el uso en sillones reciclables.
- Asegúrese de distribuir el peso simétricamente.
- Asegúrese de que todos los elevadores estén nivelados y las piernas estén bien ubicadas antes de usarlos.
- Inspeccione periódicamente el desgaste y posibles daños.

### MANTENIMIENTO

Los conos de elevación tienen un mantenimiento mínimo.

Se puede limpiar fácilmente con un paño húmedo.

