# MANUAL DE INSTRUCCIONES

## Andador CAPITEL

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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**NOTICE**

Notice to the user and/or patient: any serious incident which has occurred in connection with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

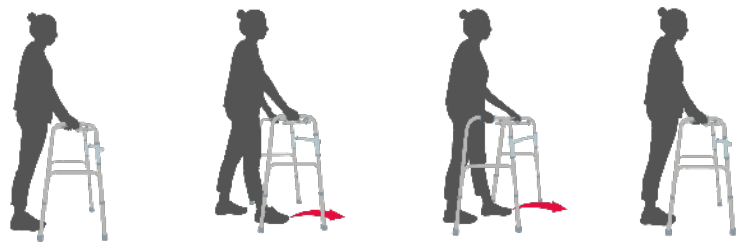
Read these instructions carefully to familiarise yourself with the walker before use.

Remove the walker from its packaging and place the legs on the floor. Unfold it by opening the side frames until you hear the safety click. Adjust the height to the correct level. To do this, press the pin with your finger, extend the legs to the desired height and fix them again. Check that the armrest and handle are correctly adjusted.

**1. HOW TO USE THE WALKER**

First, place the walker one step in front of you and make sure that the legs are at the same height. Hold on to the top of the walker by the handles for support and walk towards it.

To fold, press the locking spring on the upper front tube and fold the side frames over the front frame.

**2. TIPS AND PRECAUTIONS**

1. Ensure that the walker is adjusted to the correct height and that the locking springs are secure and protrude through the holes in all the legs at the top.
2. Fasten all screws securely and tightly. Before using the walker, check that the side frames are fully extended and locked.
3. Check that the rubber feet and handle are in good condition for use before using the walker.
4. Avoid exposing the product to sunlight.
5. Do not hold the walker by the front bar or use the front bar to sit on.
6. Do not use the walker to go up or down stairs. Remember that this product should only be used on flat, smooth surfaces.
7. Supports up to 100kg.
8. Do not apply force on one side only or the walker will tip over.
9. Do not adjust the walker during use. Assembly and disassembly of the walker should only be done when the walker is not in use.
10. Do not use escalators with the walker.
11. Do not change the direction of the walker suddenly when using the walker on a slope.
12. It should be used on level streets. Avoid bumping the walker against obstacles.
13. Do not walk with a walker uphill with an incline of more than 7 degrees. Do not walk with a walker downhill with a slope of more than 15 degrees. When walking uphill or downhill, the lateral incline or decline of the slope must not exceed 3.5 degrees.

### 3. INDICATIONS

It is designed to assist walking for the elderly, disabled and people who have difficulty walking.

Make sure that all parts are tight and that the legs and wheels are in contact with the ground before use.

### 4. CLEANING AND MAINTENANCE

1. Clean the walker regularly with a clean, dry cloth.
2. Make sure that the rubber non-slip pads on the feet are tight and secure. If they are worn, replace them immediately. Check them especially if the walker is used on slopes.
3. Store and maintain the walker in a cool, dry place and avoid excessive moisture.

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**GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD**
**EC REP** **SUNGO EUROPE B.V.**
**GRUPO R. QUERALTÓ S.A.**