# MANUAL DE INSTRUCCIONES / INSTRUCTION MANUAL / MANUALE D'UTILISATION

## Andador / Rollator / Déambulateur PICASSO

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 **MOBICLINIC**  
Spanish Leading Company Since 1985

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## ENGLISH

### NOTICE:

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

### INSTRUCTIONS FOR USE:

1. To activate the brakes press upwards on the handlebars.
2. Place both feet together in the middle of the rollator frame.
3. Deactivate the brakes by lifting the brake handlebars.
4. Grasp both handlebars of the rollator.
5. Move the rollator forward to a distance that is comfortable for you.
6. Move towards the inside of the rollator frame with your weaker leg.
7. Move forward with your more agile leg while leaning on the handlebars. Both feet should be between the rollator wheels. Continue walking repeating steps 5 and 7.

### INDICATIONS:

- It is designed for assistance to patients, elderly and/or disabled.
- It should be used indoors on a flat, smooth surface.
- Check regularly that all nuts and bolts are tightened securely, AND ensure that all four legs are level with the floor.
- Hold on to the seat from both sides to stand up.
- Maximum user weight: 100 kg
- Apply the brakes before sitting down and make sure the brake is applied before standing up to use the rollator again.
- Keep hips parallel to the front bar of the rollator.
- Make sure all wheels are on the ground before walking.

### TIPS AND PRECAUTIONS :

- The rollator must be braked if seated. Do not use as a wheelchair. It is only for temporary resting while being braked.
- Do not apply force on one side only or the rollator will tip over.
- Do not go up or down stairs or steps with the rollator.
- Do not adjust the rollator during use. Assembly and disassembly of the rollator should be done only when the rollator is not in use.
- Do not walk on conventional stairs or escalators with a rollator.

- Do not change the direction of the rollator suddenly when using the rollator on a slope.
- It must be used on level streets. Do not use on bumpy roads. Avoid hitting the rollator against obstacles.
- Do not walk with a rollator uphill with an incline of more than 7 degrees. Do not walk a rollator downhill with a slope of more than 15 degrees. When walking uphill or downhill, the lateral incline or decline of the slope should not exceed 3.5 degrees.
- Avoid turning your body to the opposite side of the rollator.
- Clean the rollator regularly with a clean, dry cloth.
- Do not use on conventional or escalators.
- Do not place heavy objects in the basket, doing so may cause the rollator to tip over.

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**GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD**
SUNGO EUROPE B.V.
Imported by/Importado por: Grupo R. Queraltó S.A.