MANUAL DE INSTRUCCIONES

Elevador WC

TITÁN SIN TAPA

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. WARNINGS

Please read these instructions carefully before using this TITAN toilet lift and keep them for reference.

The TITAN toilet lift is designed to sit on the toilet bowl and should not be used for other purposes. Do not use it as a ladder, do not stand on it and do not allow children to play with it.

Check frequently and before use that the fasteners are correctly attached. Check all parts carefully. If any of them are not in good condition, replace them with original spare parts.

2. INDICATIONS

- This product is recommended for elderly people or people with mobility difficulties who are unable to use the toilet on their own.

- Maximum user weight: 160 kg.

3. ASSEMBLY AND INSTALLATION

1. Remove all parts from the packaging and check that all parts are included.
2. Fit the rims of the riser to the bowl of your toilet and tighten the screws until it is securely in place.

4. CARE AND MAINTENANCE

- Wash and dry the lift after each use. Remove dirt with a damp cloth and mild soap and then wipe dry. Do not use harsh chemicals or abrasive cleaners to avoid damaging the lift.

- At the end of its service life, dispose of the hoist properly.

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FOSHAN SUNCARE MEDICAL PRODUCTOS CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.