MANUAL DE INSTRUCCIONES

Elevador de inodoro TAJO

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. ADVICE AND PRECAUTIONS

The TAJO toilet lift is designed to sit on the toilet bowl and should not be used for other purposes. Do not use it as a ladder, do not stand on it and do not allow children to play with it.

Check frequently and before use that the fasteners are correctly fixed. Check all parts carefully. If any of them are not in good condition, replace them with original spare parts.

2. INDICATIONS

This product is recommended for elderly people or people with mobility difficulties who are unable to use the toilet on their own.

Maximum user weight: 100kg.

3. ASSEMBLY AND INSTALLATION

1. Remove all parts from the packaging and check that all parts are included.
2. Take one of the armrests and place it in the side hole on the back of the lift. Insert and screw the screw between the armrests and the lift and then repeat the process with the other armrest.
3. Fit the three edges of the riser to the bowl of your toilet and tighten the screws until it is secure.

4. CARE AND MAINTENANCE

Wash and dry the lift after each use. Remove dirt with a damp cloth and mild soap and then wipe dry. Do not use harsh chemicals or abrasive cleaners to avoid damaging the lift.

At the end of its service life, dispose of the hoist properly.

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FOSHAN SUNCARE MEDICAL PRODUCTS CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.