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IMPORTANT

Keep these instructions for future reference. Read these instructions before using this product.

Thank you for choosing the MOBI 1 anti-skid mattress! In this instruction manual you will find all the information you need to assemble, use and maintain the wheelchair safely. Please read all instructions carefully before using the product.

Follow the recommendations in this manual, as they are designed to improve safety and performance.

NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

This device consists of two parts: the mattress and the motor. Thanks to its small compressor, the motor generates little noise and consumes little energy. The control panel is simple and very easy to use. The mattress relieves pressure through its alternating air cells that inflate and deflate approximately every 6 minutes. Constant pressure on a bony prominence is known to be the main cause of bedsores. The continuous movement of the mattress decreases the constant pressure and improves circulation.

INDICATIONS

This product is intended for:

- To assist and reduce the incidence of pressure ulcers by optimising patient comfort.
- For long-term home care of patients suffering from pressure ulcers.
- If necessary, replace the fuse with the one indicated: T1A 250V. Input current: ~220V 50Hz, 0.1A. Rated weight 297,62 lbs.

CONTRAINDICATIONS

- This equipment is not suitable for use in the presence of a flammable anaesthetic mixture with air or with pure oxygen or nitrous oxide.
- Do not turn or roll patients with a bone fracture or spinal injury.
- Do not smoke near the compressor.


- Do not bring the compressor close to hot surfaces.
- There is a risk of explosion if used near flammable anaesthetics, of manual brakes, wheels and weight limit.

1. PRODUCT FEATURES

PRODUCT NAME	Anti-frost mattress
MODEL	MOBI 1
MODEL REFERENCE	A-00050/MB1
MAXIMUM SUPPORTED WEIGHT	297,62 lbs
PRODUCT DIMENSIONS	6,49"*10,82"*6,88"

2. INSTRUCTIONS FOR USE

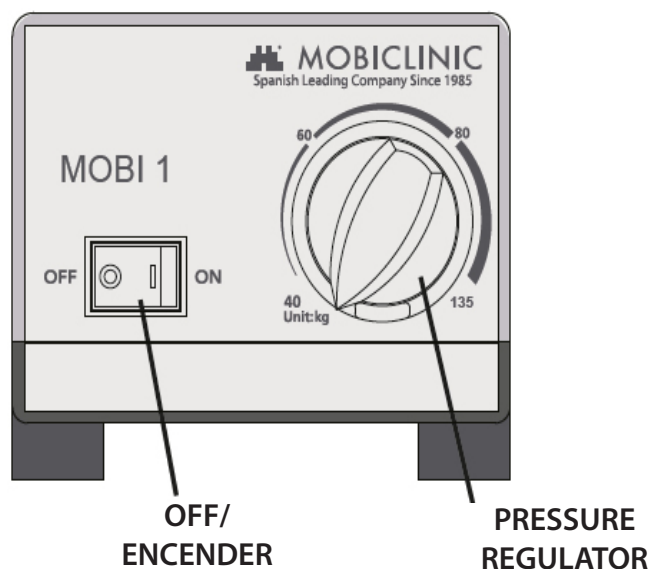1. Place the mattress on the bed with the air sleeve towards the feet.
2. Use the integrated hooks to hang the compressor securely on the footboard of the bed. You can also place it on a flat surface.
3. Connect the air hoses of the mattress to the compressor.
4. Connect the compressor to an electrical outlet and ensure that the cable is not in danger of damage.
5. Turn on the switch on the compressor control panel to start inflating the mattress.
6. Once inflated, adjust the mattress using the compressor knob.

3. MAINTENANCE AND CLEANING

1. Make sure that the system is disconnected from the mains before cleaning.
2. Do not immerse in water.
3. Do not dry clean.
4. Do not use bleach or products containing bleach.
5. Do not iron.
6. Do not autoclave or use phenol for the cleaning procedure.
7. Do not wash air cells, clean with mild soapy water or disinfectant.
8. Clean the compressor with a soft dry cloth.
9. Operating conditions:
 - Ambient temperature: 10°C to 40°C.
 - Relative humidity: 10% to 75%.
10. Storage and shipping conditions:
 - Ambient temperature: From -18°C to 43°C.
 - Relative humidity: 10% to 95%.

4. CONDITIONS OF GUARANTEE

This product is covered by a 36 month warranty against manufacturing defects.

Thank you very much for trusting Mobiclinic with this MOBI 1 anti-squeeze mattress! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or need further assistance, please do not hesitate to contact us through our customer service at clientes@mobiclinic.com . We appreciate your comments and suggestions for further improvement.

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FACTORY CO., LTD
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.