MANUAL DE INSTRUCCIONES

Andador rollator COMPOSTELA

| | |
|-------------------------|-----|
| Manual de instrucciones | ES |
| Instruction manual | EN |
| Manuel D'Utilisation | FR |
| Manuale D'Istruzioni | IT |
| Anweisungen | DE |
| Instruções | PT |
| Gebruiksaanwijzing | NL |
| Bruksanvisning | SWE |
| Instrukcja obsługi | PL |
| Brugsanvisning | DK |

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NOTICE

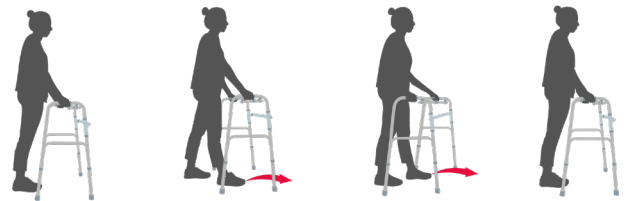
Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

Please read these instructions carefully to familiarise yourself with the walker before use.

1. Remove the walker from its packaging and place the legs on the floor.
2. To unfold, open the side frames until you hear the safety click.
3. Adjust the frame to the correct height. To do this, press the pin with your finger, extend the legs to the desired height and fix it again.
4. Check that the armrest and handlebars are securely fastened and be careful not to hurt yourself if they are loose.
5. Make sure that the rubber non-slip pads on the feet are tight and secure. If they are worn, replace them immediately. Check them especially if the walker is used on slopes.

1. HOW TO USE THE WALKER

First, place the walker one step in front of you and make sure that the legs are at the same height. Hold on to the top of the walker by the handles for support and walk towards it. To turn, take small steps. To fold, press the locking spring on the upper front tube and fold the side frames towards the front frame.

2. WARNINGS AND PRECAUTIONS

1. Make sure the walker is adjusted to the correct height and check that the locking springs are fixed and protrude through the holes in all the legs at the top.
2. Fasten all screws securely and tightly.
3. Before using the walker, check that the side frames are fully extended and locked.
4. Check that the rubber mats and the rubber handle are in good condition for use before using the walker.
5. Avoid exposing the product to sunlight. Store and maintain the walker in a cool, dry place and avoid excessive moisture.
6. Do not hold the walker by the front bar or use it to sit on.
7. Do not use the walker to go up or down stairs. Supports up to 100 kg.

3. INDICATIONS

1. The walker can be used to assist the elderly in walking and shopping, and can be used by users to carry, rest and assist in walking.
2. The weight of the user does not exceed 110 kg, otherwise it will be deformed. The weight of the shopping basket does not exceed 10 kg.

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FOSHAN KAIYANG MEDICAL TECHNOLOGY GROUP CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.