MANUAL DE INSTRUCCIONES

Silla de ruedas bariátrica PLUS 225

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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NOTICE

Notice to the user and/or patient: any serious incident which has occurred in connection with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

1. PRODUCT STRUCTURE

1. Handle
2. Backrest
3. Folding armrests
4. Armrest frame
5. Side panel
6. Rear wheel set (with Quick Release)
7. Handle
8. Side brake
9. Front wheel
10. Adjustable bar
11. Footrest
12. Anti-tip wheels
13. Seat (60cm wide)

2. PRODUCT SCOPE AND APPLICABLE PERSONS

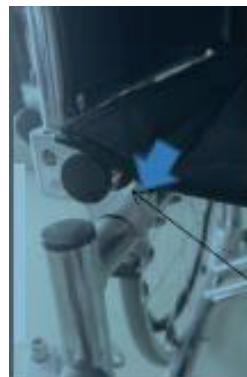
The wheelchair is used as a mobility tool for disabled people, patients and elderly people who have difficulties in moving.

Beginners should be accompanied by assistants to avoid accidents due to improper handling.

The maximum weight for this bariatric wheelchair model is 225kg.

3. INSTALLATION AND METHOD OF USE**1. Folding and unfolding of the wheelchair**

Folding: As shown in the left figure, first of all, fold the both side foot treadles outwards, then, lift the middle part of the seat cushion by hands to fold the whole wheelchair.

Expansion:

1. Stand on the rear end of the wheelchair to unfold it towards left and right sides.
2. Open the wheelchair safely until the H shaped card gets stuck in the jack catch (as shown in the left figure).

2. Handle operation

As shown in the left figure, press the button towards the outside to place the handle (as shown in the right figure).

1. Hold the handrail tightly to push it towards arrow direction (as shown in the foregoing figure).
2. Press the handrail tightly to make support when the handle is pulled upwards. When the handle is locked and fixed, you will hear click sound.

3. Backward handrail operation

Press the button towards the body outside, then, pick up the handrail directly (as shown in the left figure).

As shown in the foregoing figure, slightly press the handrail according to the arrow direction. When there is click sound, the handrail will be locked and fixed.

4. The usage of movable detachable foot supports

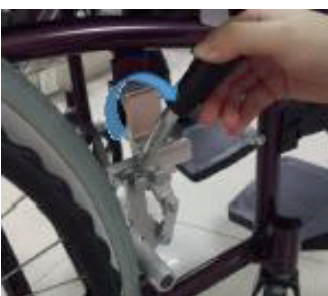
As shown in the left figure, pull the spring locking plate towards the body, then, rotate the detachable foot support (the direction as shown) to dismount the movable foot supports.

As shown in the foregoing figure, please make rotation towards the inside, then, the detachable foot support will be locked after hearing the click sound.

5. Brake operation

As shown in the left figure, the brake will be locked after brake hand shank is pressed forwards (the arrow direction as shown). The brake will be released after pressing operation in the reverse direction.

6. Anti-tip wheel

As shown in the left figure, please pick up the foot treadle and apply the supportive wrench to release the screw, then, adjust it to the suitable height according to the users' demand.

Release the screw according to the arrow direction (standard tools).

7. Quick releasing wheel operation

As shown in the left figure, the front wheel will be dismounted after pressing the black button in the middle of a wheel by hands, similarly, the rear wheel can be installed when the button is pressed and pushed inwards.

4. STORAGE CONDITIONS AND STORAGE METHODS

The manual propelled wheelchair shall be stored in the room without corrosive gas, favorable ventilation and cleaning environment after packaging, what's more, the direct sunshine shall be avoided!

5. MAINTENANCE METHOD

1) Check whether the wheelchair tire is inflated normally. If the air volume is insufficient, please inflate it in time. The method of inflation is the same as that of bicycle tires. If a puncture occurs, please repair it at the nearest bicycle repair shop.

2) After the wheelchair has been used for a period of time, it is necessary to detect whether each part of the wheelchair is maneuverable and whether the bolts are loose. If there is any looseness, it must be adjusted in time to avoid hidden dangers of insecurity.

3) Lubricating oil should be added to moving parts every week to prevent inflexible use.

4) After using the wheelchair, use a soft dry cloth to wipe off moisture and pollution on its surface to prevent rust.

5) Seat cushions and backrest cushions should be kept clean and clean to prevent skin infections.

6. PRECAUTIONS AND WARNING MESSAGES FOR USING WHEELCHAIRS

- 1) The wheelchair shall be used after brake operation is confirmed to be normal.
- 2) Please don't step on the foot treadle when you get away from wheelchair to avoid forwarding dumping.
- 3) Please don't place the wheelchair in the vicinity of inflammables to avoid user injury caused by backrest combustion.
- 4) When the wheelchair needs to climb the steps, the caregiver shall step on the foot tube, then, press the handrail downwards to make the front wheel go forward towards the stair upward side, after that, pull the handrail upwards to enhance the rear wheel, during that period, please avoid push the wheelchair by force on the steps.
- 5) If the wheelchair shall be used on the inclined slope, it is necessary to ask for others to provide assistance to avoid wheelchair dumping. The wheelchair shall move towards the upward side of inclined slope when the wheelchair needs to be driven on the inclined slope, at this moment, the facilitator shall support in the wheelchair backrest to push it move downwards with the slow speed.
- 6) When the user sit at the wheelchair, please don't hold the handle to uplift the wheelchair directly as long as the wheelchair applies folding backrest because it is likely to be destroyed in such case.

7. LABELS, PACKING LOGO DESIGN

SYMBOL	INTRODUCTIONS	SYMBOL	INTRODUCTIONS
	Batch Code		Keep dry
	Warnings and Precautions		Non sterile
	Medical device		Manufacture Date
	Manufacturer Name Address		Name and Address of European Union Representative
	CE Symbol		Used-by date

	<p>Consult instructions for use</p>		<p>Unique Device Identification</p>
	<p>Country of Manufacture</p>		<p>Model Number</p>
	<p>Importer</p>		<p>Distributor</p>

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GUANGDONG KAIYANG MEDICAL TECHNOLOGY GROUP CO., LTD.
EC REP SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.