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**IMPORTANT**

Keep these instructions for future reference. Read these instructions before assembling and using this product. Adult assembly is required.

Thank you for choosing the Denver wheelchair! In this instruction manual you will find all the information you need to assemble, use and maintain the wheelchair safely. Please read all instructions carefully before using the product.

Follow the recommendations in this manual, as they are designed to improve safety and performance.

**NOTICE**

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

**1. PRODUCT FEATURES**

<b>PRODUCT NAME</b>	Wheelchair
<b>MODEL</b>	DENVER
<b>MODEL REFERENCE</b>	A-00461/07
<b>MAXIMUM SUPPORTED WEIGHT</b>	242,50 lbs
<b>NET PRODUCT WEIGHT</b>	38,58 lbs
<b>PRODUCT DIMENSIONS</b>	31,88"*10,62"*31,49"

## 2. PARTS

1. Support	8. Handlebars
2. Armrests	9. Rear upper tube
3. Handle tube	10. Nut
4. Seat	11. Brake
5. Footrest support	12. Rear wheel
6. Supporting tape	13. Main structure
7. Footrest	14. Front wheel

## 3. INDICATIONS AND CONTRAINDICATIONS

### INDICATIONS

- The wheelchair is used as a mobility tool for the disabled, patients and the elderly who have difficulties in action.
- It is advisable that they are always accompanied at the beginning by a partner to prevent possible accidents due to inexperience in driving.

## CONTRAINDICATIONS


- The wheelchair can be used normally on asphalt and concrete floors. When driving on uneven terrain or soft ground, drive slowly. Avoid riding on wet grass as the wheels may skid.
- When driving on sloping ground, do not turn the wheelchair or change direction abruptly.
- Do not brake going downhill to avoid falling forward.
- Do not use the wheelchair on dangerous sloping ground (snow, grass, wet leaves, etc.).
- The brake is used to sit, stand or stabilise the wheelchair. Do not use it when moving, especially when going downhill, to avoid the risk of tipping over. If unaccompanied, hold the wheels firmly and brake gradually.
- When sitting or getting up from the chair, the footrests must be turned upwards. Do not stand on them; the chair may tip over.
- When you are in the chair, do not lean forward to reach for something. Do not bend over to pick things up from the floor with your hands between your knees.
- When transferring from the wheelchair to another seat, lean as far back as possible to prevent the wheelchair from falling forward and damage to the bolts or seat.
- Do not use the wheelchair in moving vehicles.
- Keep the chair away from fire or flammable objects.
- Do not make any modifications to the wheelchair without the consent of an authorised technician.
- Do not use the wheelchair on stairs or escalators.
- Environmental operating conditions:
  - Temperature -25°C~50°C
  - Relative humidity: ≤80%
  - Atmospheric pressure: 86kPa~106kPa

## 4. ASSEMBLY INSTRUCTIONS

1. **Install the rear top tube to the frame:** Insert the hex bolts through the frame and rear tube, front to back, and tighten the nut.

2. **Install the legrests to the frame:** Insert the bracket into the frame and pass the C-buckle through the frame and the front tube of the footrest.

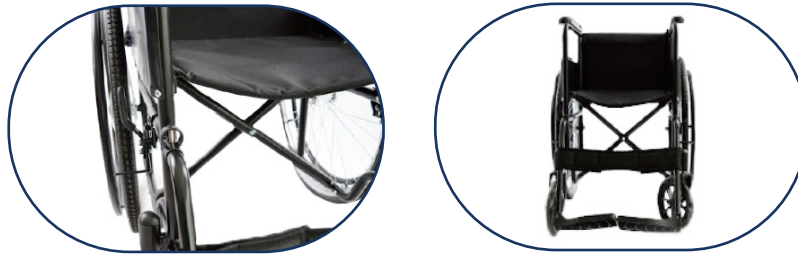3. **Install the tape around the footrest tubes.**


## 5. INSTRUCTIONS FOR USE

1. The wheelchair can be used with the assistance of a companion or by the user independently.


2. **Brake use:** When the wheelchair is stationary, the user should hold the left and right brakes with both hands and, at the same time, push the brake handles forward to lock the rear wheels.

3. **Adjusting the height of the footrest:** First loosen the nut with a spanner and remove the screw. Pull the footrest tube down and adjust the footrest to the desired height. Then align the holes to install the screw and tighten it with the wrench.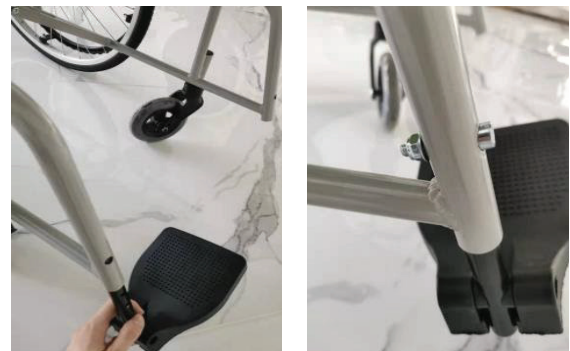

**4. Unfold the wheelchair:** Hold the handlebars to spread the two sides slightly apart. On each side, press down on the seat tubes with the palm of your hand until the seat tubes snap between the brackets and the wheelchair is unfolded. Do not pull on the sides of the frame with force to avoid damage. When pressing on the seat, do not place your fingers between the tubes to avoid pinching your fingers.

**5. Folding the wheelchair:** Swing up the footrests and carefully lift both sides of the seat with both hands. If necessary, the movable legs and the rear upper tube can be removed.

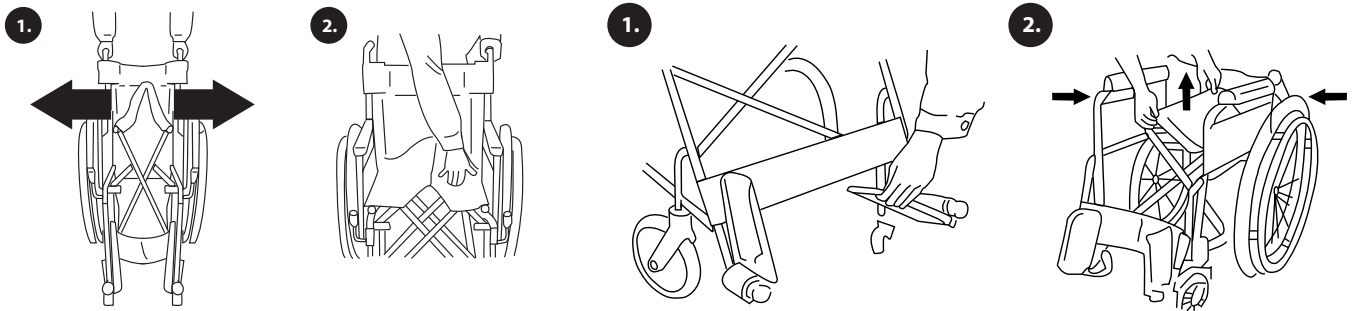## 6. MAINTENANCE AND CLEANING

Regular inspection and maintenance should be carried out in accordance with the following procedures:

1. Periodically check the following points:

- The wheels are not cracked.
- The armrest bracket is secure, easy to remove and the screws are tightened securely.
- The adjustable height works and locks securely.
- The seat and backrest are not damaged, deformed or loose.
- The armrest cushions are comfortable.
- Metal parts are not rusted or damaged.

2. It is recommended that it is checked once a month and thoroughly checked every 6 months.

3. The seat, backrest, handlebars and armrests can be cleaned with a clean, damp cloth. Do not use solvents to clean the chair.

4. Store the chair in a place protected from water and rain to prevent premature rusting of the metal parts.

5. Conditions for storage:

- Ambient temperature:  $-40^{\circ}\text{C} \sim 65^{\circ}\text{C}$
- Relative humidity:  $\leq 80\%$
- Atmospheric pressure:  $86\text{kPa} \sim 106\text{kPa}$

## 7. CONDITIONS OF GUARANTEE

This product is covered by a 36 months warranty against manufacturing defects.

Thank you very much for trusting Mobiclinic with this DENVER wheelchair! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or need additional assistance, please do not hesitate to contact us through our customer service at **clientes@mobiclinic.com**. We appreciate your comments and suggestions for further improvement.

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INTCO MEDICAL (HK) CO., LTD
GRUPO R. QUERALTÓ S.A.