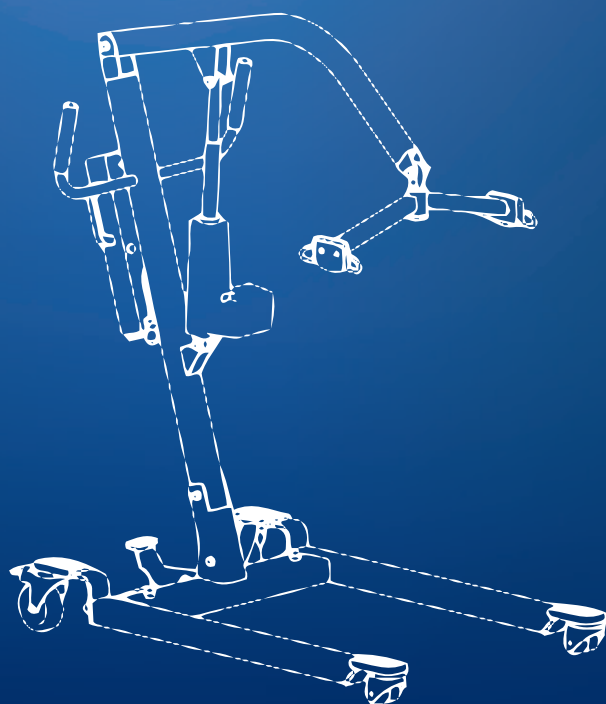
# MANUAL DE INSTRUCCIONES

Grúa

FORTUNA BASIC

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SWE

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PL

Brugsanvisning

DK

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TO KEEP FOR FUTURE REFERENCE.

### 1. SECURITY

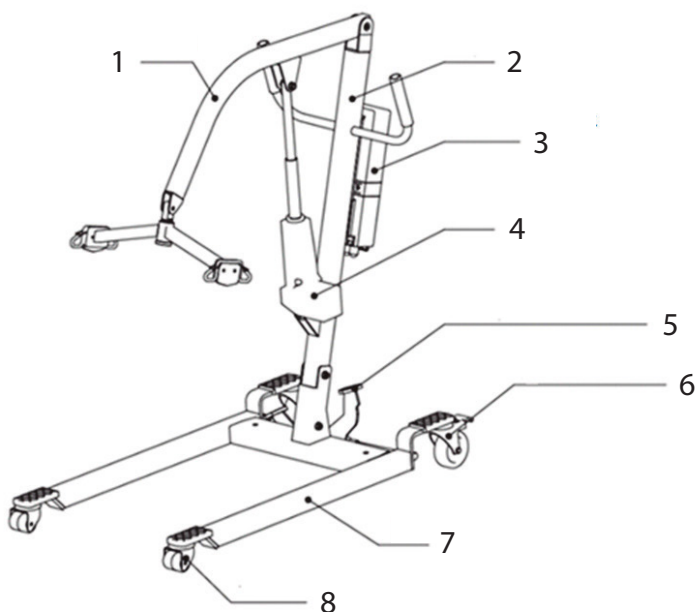
- Read this instruction manual carefully.
- Mount the product in accordance with this instruction manual.
- Mount the lift actuator properly.
- Charge the battery for at least 4 hours until fully charged.
- For safety reasons and according to the user's needs, select the appropriate model, size, material and accessories.

### NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.


### 2. PARTS OF THE CRANE

1. Crane arm
2. Structure
3. Control box
4. Actuator
5. Foot pedal for width adjustment
6. Rear wheels with 4" brakes
7. Base
8. 3" front wheels

### 3. SPECIFICATIONS

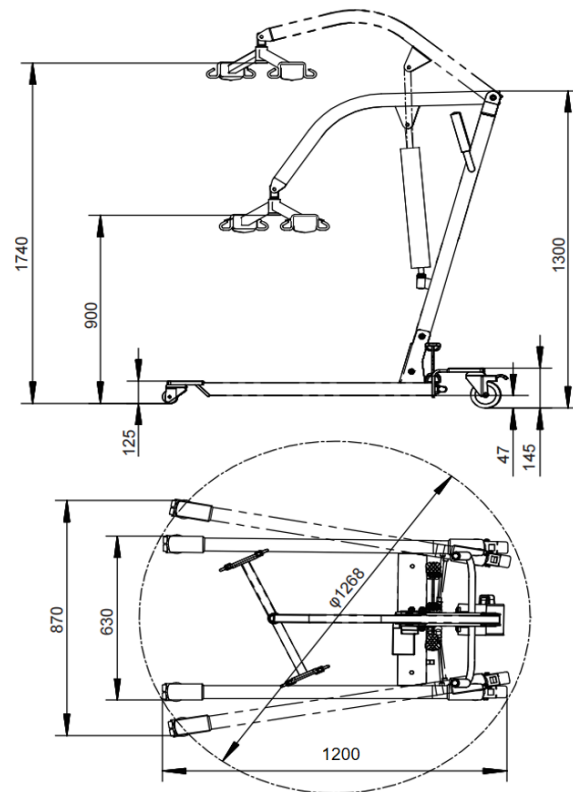
1. Product size: 122x63x130 cm
2. Maximum reach of the fork: 87 cm
3. Lifting range: 900 ± 10 mm (lower), 1740 ± 10 mm (highest)
4. Input power: DC24V
5. Motor: 24V/6000N
6. Load capacity: 180kg
7. Net weight: 38kg
8. Noise: ≤ 53dB
9. Built-in battery for emergency use, Full charge can lift times: 5-8 times

### 4. INSTALLATION
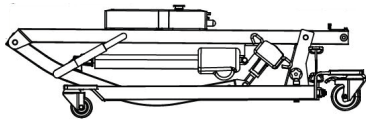
#### 4.1. PARTS

Please check if any parts are missing (the list of accessories will be subject to the list enclosed with the product). In case of damage or omissions, please contact us or your authorised dealers.


DESCRIPTION	QUANTITY	IMAGE
Crane	1 piece	
Control box	1 piece	
Actuator	1 piece	

Remote + Cable	1 set	
Wrench #16 and #17	1 set	
Instruction manual	1 piece	

## 4.2. ASSEMBLY PROCESS

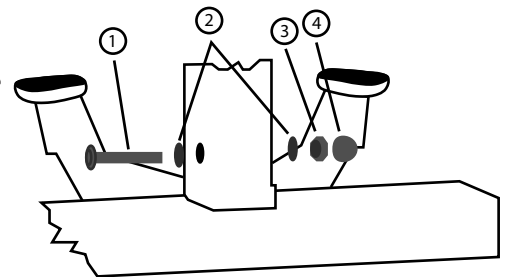
**STEP 1:** Lift the frame and align the holes. Fix the M10x75 hex bolt as shown in the picture and fix the nut correctly (make sure that the frame does not move).

- 1 - Hexagonal screw M10x75
- 2 - Washer
- 3 - Nylon lock nut M10
- 4 - Nut cap


**STEP 2:** Insert the lift arm into the upper slot of the frame. Make sure the holes are aligned, then fasten the M10x60 hex screw as shown in the picture on the right and fix the nut cover (no movement is allowed on the lift arm).

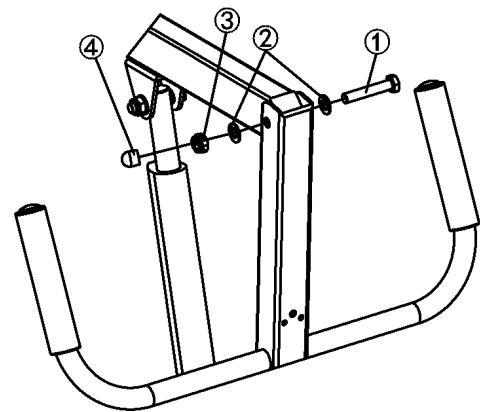- 1 - Hexagonal screw M10x60
- 2 - Washer
- 3 - Nylon lock nut M10
- 4 - Nut cap


**STEP 3:** Align the holes and fix the actuator. Hook in the M10x60 hexagonal screw as shown in the drawings and fix the nut cover correctly.

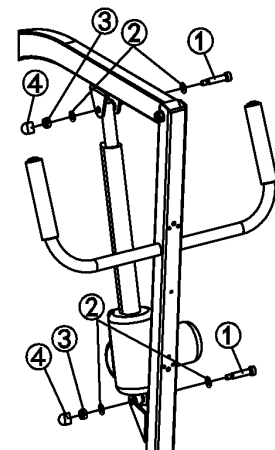- 1 - Hexagonal screw M10x60
- 2 - Washer
- 3 - Nylon lock nut M10
- 4 - Nut cap

**STEP 4:** Installing the control box. Align the holes of the control box with its fixing plate as shown in the figure, and fix the screws, washers and nuts in sequence.

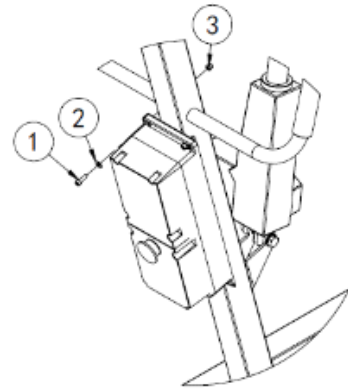
- 1 - Tornillo M5x20
- 2 - Arandela
- 3 - M5 nylon connector

## 5. USER GUIDE

### 5.1. USE OF CONTROL

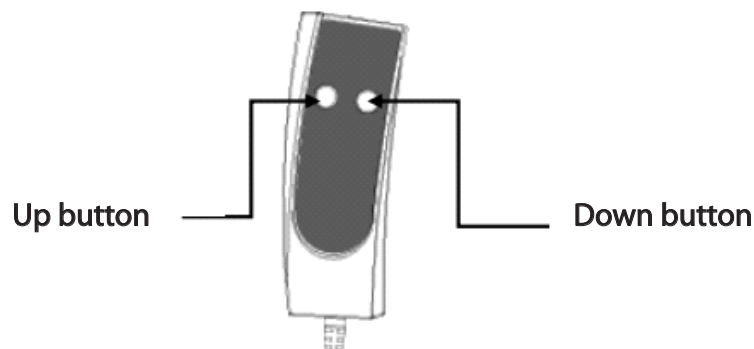
Control raising and lowering of the lift arm using the up and down buttons on the hand control. The lift stops operating when the up/down button is released.

### 5.2. BASE WIDTH ADJUSTMENT

The width of the base can be adjusted by pedals.

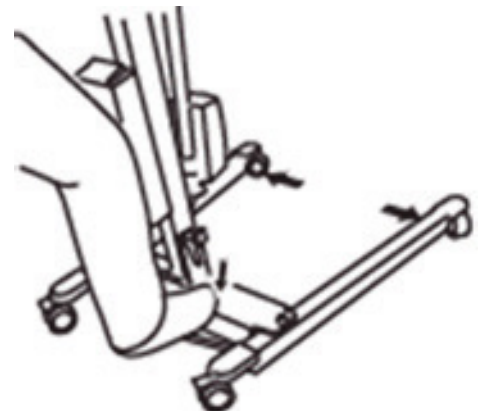Step on the left pedal for a wider base and step on the right pedal for a narrower base.


### 5.3. EMERGENCY STOP BUTTON

The emergency stop button is used to switch the power on and off. It allows the motor to stop running immediately in case of danger.


Switching off: Press the red button (see picture A), the control device is suspended.

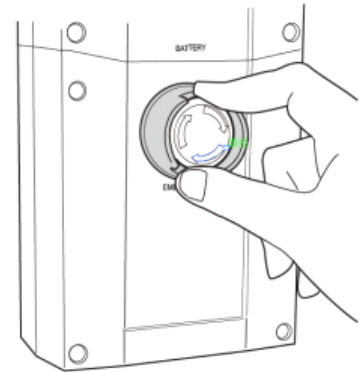Switching on: Turn the red button clockwise (see picture B) and the control device is activated.

### 5.4. CONNECTION OF THE CONTROL DEVICE

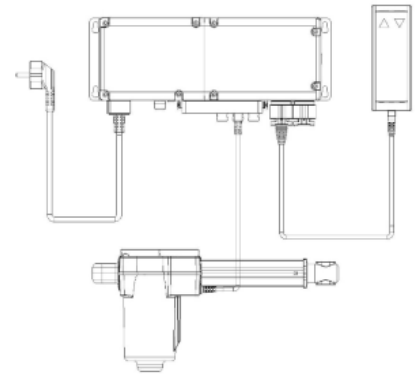Plug the power cable, the actuator plug and also the plug for the handle control into the control box as shown in the figure on the right).


### 5.5. REAR WHEEL LOCKING AND UNLOCKING

The rear wheels can be locked to prevent them from moving. Use your foot to lock and unlock the rear wheels.

Caution: It is not necessary to lock the rear wheels for most lifting and lowering movements. We suggest only locking the wheels when the lift moves and may cause personal injury.

#### **WARNING**

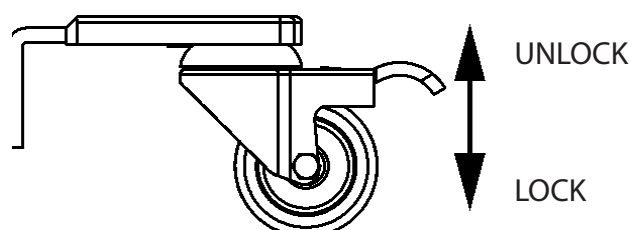Blocking the actual wheels of the crane increases the risk of tipping during the lifting and lowering movement.


## 5.6. CRANE OPERATION

### 5.6.1. SAFETY OPERATION

Keep the lift arm and body in an upright position.

Attach 2 sling loops that are for the back support to one end of the hook, and the other 2 loops are attached to the other end of the hook after making a cross and placing them under the leg. (The length of the loop is determined by the user's height).

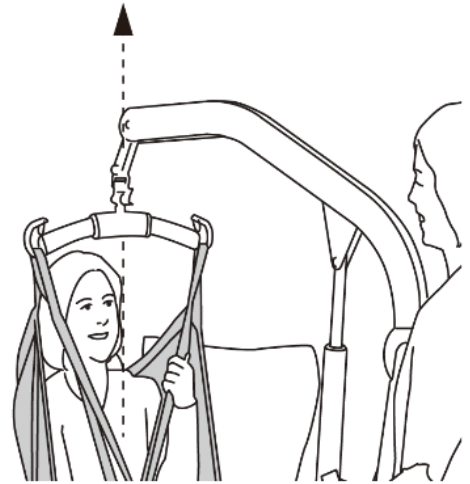
For details, see "Lift Location". Make sure that the lift works properly each time.

**! WARNING**

An unbalanced lift creates the risk of rollover.
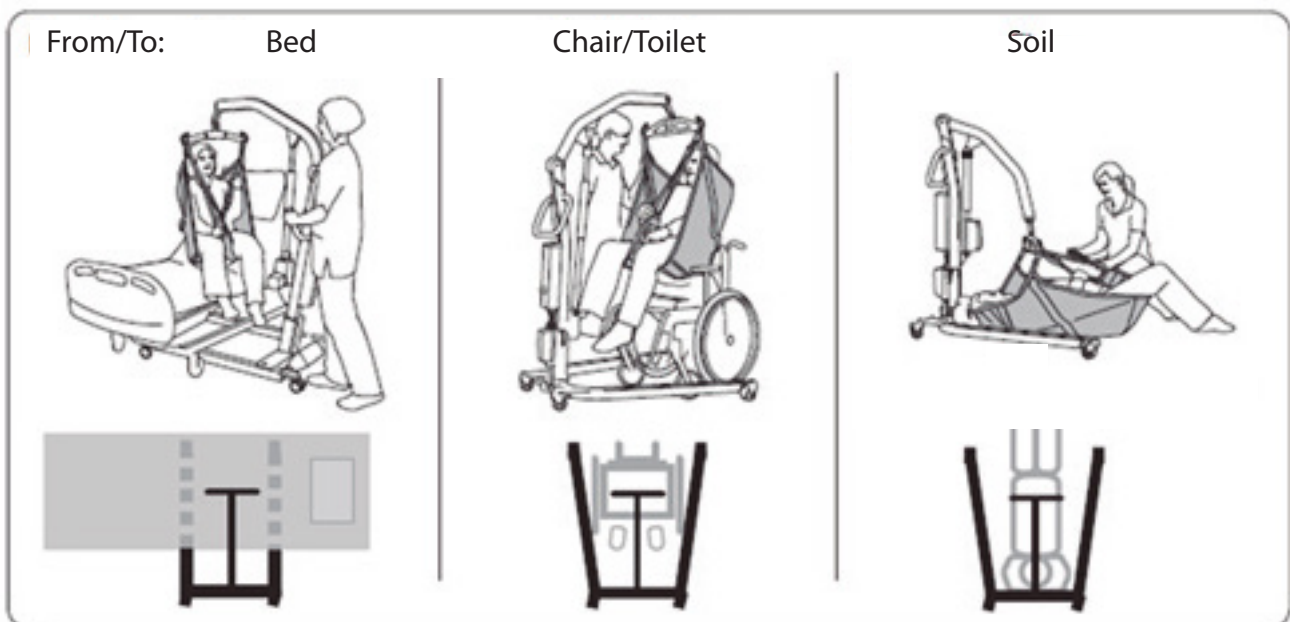
### 5.6.2. USES OF THE PATIENT TRANSFER CRANE

NOTE: A pillow under the patient's head will provide additional comfort.

## 6. CARE AND MAINTENANCE

The routine inspections listed below shall be carried out daily in the interests of safety.

1. Check for any damage or wear by visual inspection.
2. Ensure that the lift arm bolts and nuts are securely fastened.
3. Check if the lift arm hook is working properly.
4. Check the controller, actuator and base width adjustment.
5. Check emergency lowering function (both manual and electric).

If necessary, it is forbidden to clean the crane with chemicals containing phenol and chlorine, which may cause damage to the plastic powder or plastic material.

Transport and storage:

Press the emergency stop button to transport or not to use the lift for a certain period of time. The storage temperature of the crane should be above 0°C and relative humidity ≤60%.

Inspection:

Carry out a complete inspection at least once a year. Pay particular attention to worn parts, such as connections and screws.

## 7. PROBLEM SOLVING

DESCRIPTION	PROBLEM SOLVING
Lift does not work (lifting and lowering)	<ol style="list-style-type: none"> <li>1. Check if the emergency stop button is pressed.</li> <li>2. Check that the controller and actuator cables are properly connected.</li> <li>3. Check the battery level on the control box display.</li> <li>4. Check for any problems or damage to the battery contact panel.</li> <li>5. Request after-sales service if the lift still does not work.</li> </ol>

Charger cannot charge the battery	<ol style="list-style-type: none"> <li>1. Check if the charger is fully plugged into the mains socket.</li> <li>2. Check if there is any problem or damage to the battery contact panel.</li> <li>3. Request after-sales service if the lift still does not work.</li> </ol>
The crane stops when ascending	<ol style="list-style-type: none"> <li>1. Check if the emergency stop button is pressed.</li> <li>2. Use the emergency lowering device (see user's guide).</li> <li>3. Call for service if the lift still does not work.</li> </ol>

## 8. GUARANTEE

### 8.1. GUARANTEE CONDITIONS

From ex-factory date:

1. Control box: 6 months free replacement and 1 year repair warranty.
2. Battery: free replacement if battery capacity is less than 60% of nominal within half a year.
3. Other parts such as wheels, pedals and plastic parts: no warranty. In case of damage during use, please contact the dealer.
4. Hi-Fortune will be liable for all faults if they have to be replaced or repaired in the case of manufacturing defects.
5. After 1 year a paid service will be provided.

### 8.2. EXCLUSIONS FROM THE GUARANTEE

1. Damage caused by improper use and maintenance.
2. Damaged parts due to failure to use standard manufacturer's spare parts.
3. Parts damaged by accident.
4. Warranty is not provided for wear part such as wheels, sling, etc.
5. Never change the setting of the patient lift without permission.
6. Damage caused by force majeure such as typhoons, floods, earthquakes, etc.

## 9. INDICATIONS AND CONTRAINDICATIONS

### Indications:

All patients with limited mobility not mentioned in the contraindications.

### Contraindications:

Disorders such as Osteogenesis imperfecta, osteoporosis or damage to the spine and mental aberration or epileptic seizures may be contraindications. Use is only permitted after careful consideration of the individual patient by the physician and nursing staff.

In the meantime, the use of a patient lift should be avoided if the patient is agitated, resistant or combative.

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KUNSHAN HI-FORTUNE HEALTH PRODUCTS CO. LTD.
GRUPO R. QUERALTÓ S.A.