MANUAL DE INSTRUCCIONES/INSTRUCTION MANUAL/MANUEL D'UTILISATION

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ENGLISH

Please read carefully this direction to know your new raised toilet seat Cibeles well before use. This model has a weight capacity of 100kg. Always check to make sure that the raised toilet seat is properly and securely locked in place before using. Make certain that the knob that secures the raised toilet seat to the commode is tight at all times.

The Cibeles raised toilet seat has been designed to accommodate a wide variety of toilet bowl types. However, certain bowl types will require the removal of the existing seat and/or lid to ensure proper fit.

Assembly

1. Lift the seat and lid of the toilet.
2. Rotate adjustment knob of the raised toilet seat counter clockwise until the pad is far enough back to clear the inner rim of the bowl.
3. Place the raised toilet seat onto the bowl rim hooking the rear wings under edge of bowl. Check that the unit rests at on the bowl rim. The knob must be at the front end of the toilet bowl.
4. Center the unit on the bowl rim and push the back as far as possible with your hands.
5. Rotate adjustment knob clockwise to tighten. Turn until resistance is felt, then turn an additional 0.32 cm until snug.

Use

1. Carefully back up to the raised toilet seat until you feel the back of your knees touching it.
2. Reach back for both armrests to guide you into position.
3. Slowly lower yourself onto the raised toilet seat while bending your elbows and knees.
4. Carefully use the armrests for assistance.
5. Slowly raise yourself off of the toilet seat.
6. Ensure that you have your balance before releasing your grasp from the raised toilet seat.

Clean the seat at least once a week, using a non-abrasive detergent or non-abrasive cleaner with warm water.

Warning

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

Indications

This product is recommended for elderly people or people with mobility difficulties who are unable to use the toilet on their own.

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Imported by/Importado por: Grupo R. Queraltó S.A.